***Ventricular catheter***  
***Peritoneal catheter***

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Mode d'emploi

Instructions for use

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Instructions for use

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## Indikation

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**CAUTION**

**Federal law restricts this device to sale by or on order of a physician!**

**Description**

The *catheter* is used for draining cerebrospinal fluid (CSF) from the ventricles into an appropriate cavity. Due to the dimensions (inner diameter 1.2 mm and outer diameter 2.5 mm) the pressure-flow-characteristic of the shunt system is only an insignificant influence.

**Ventricular catheter**

The *ventricular catheters* of the Christoph Miethke GmbH & Co. KG are silicone catheters added with 11-20% BaSO<sub>4</sub>. Therefore a good identification on X-ray image is guaranteed. For a safe placement the *ventricular catheter* has got markings of 3, 5, 7, 10 and 13 cm seen from the catheter tip.

By using the *ventricular catheter* of the Christoph Miethke GmbH & Co. KG the introducing stylet can be pushed into the tip of the catheter (Fig. 1). Therefore kinking of the catheter tip is avoided. The holes start at 3 mm from the tip and end after 14 mm. In treatment of slitlike ventricles drainage from the ventricles into the brain is therefore avoided.

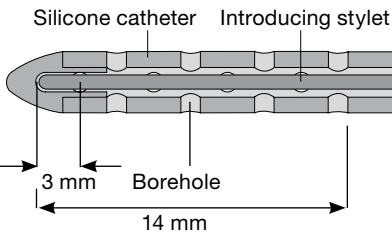
Fig. 1: Ventricular catheter

**THOMALE CATHETER**

The *THOMALE CATHETER* is a ventricular catheter with 6 holes next to the tip of the catheter. It is used preferably in the treatment in case of narrow ventricles.

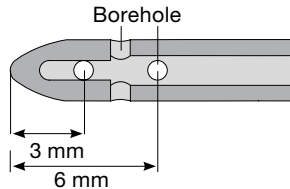
Fig. 2: THOMALE CATHETER

**Peritoneal catheter**

The *peritoneal catheter* of the Christoph Miethke GmbH & Co. KG are silicone catheters with 11-20% BaSO<sub>4</sub>. Therefore a good identification on X-ray image is warranted.

**Peritoneal catheter, striped**

The *peritoneal catheter, striped* is a silicone catheter with a stripe of BaSO<sub>4</sub> which is completely enclosed by silicone (Fig. 3).

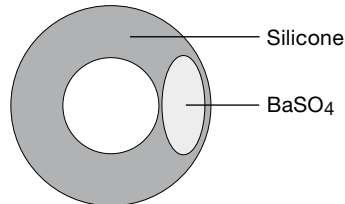
Fig. 3: Schematic cross section of the peritoneal catheter, striped

**Implantation and reimplantation**

When implanting a *catheter*, the physician have to ensure that the *catheter* is secured with ligatures. The *ventricular catheter* can be implanted with the aid of an introducing stylet. The length of the catheter that extends into the cranium can be pre-determined with the help of the *deflector*, after which it can be brought through the 90° angle. (The *deflector* can be ordered separately also).

Under no circumstances should products that have had previously been implanted in a patient be subsequently reimplanted in another, because a successful decontamination of the device cannot be reached without functional degradation.

**Tolerability to diagnostic procedures**

All *catheters* manufactured by Christoph Miethke GmbH & Co. KG are made of silicone. MRI examinations with field strengths of up to 3.0 tesla and CT examinations can be carried without endangering or impairing the functionality of the shunt. The catheters are MR Safe.

**Safety measures**

Following implantation of a shunt system, the patient's condition should be carefully and thoroughly monitored. Inflamed skin and tension in the area of the drainage tissues could possibly be a sign of infection in the shunt system. Symptoms such as headache, dizziness, disorientation, or vomiting often occur in cases of shunt dysfunction. In the event such symptoms occur, or if there is any leakage in the shunt system, either individual shunt system components or the entire shunt system must be replaced without delay.

**Sterilisation**

All products are carefully and thoroughly steam sterilized. Owing to the fact that the product is packaged in two layers of sterile packaging, five years of sterility are guaranteed. The expiration date for each item is indicated on the package. If the packaging is damaged in any way, the product should not be used under any circumstances.

**Resterilisation**

The functional safety and reliability of resterilized products cannot be guaranteed.

**Requirements of the MDD 93/42/EEC**

The MDD calls for the comprehensive documentation of the whereabouts of medical products that are applied in human beings, especially the whereabouts of implants. For this reason, the individual identification numbers of any implanted valves are to be noted in patients' records, so that in the event of any inquiries, the implant can be traced without any difficulties. Each valve is outfitted with a sticker for this purpose.

**Note on the instructions for use**

The descriptions and explanations given in this document are based on the clinical experience available up to date. It is for the surgeon's to decide if surgical procedures should be changed according to his or her experience and to surgical practice.

**Medical products consultant**

In compliance with the requirements of the European law MDD 93/42/EEC, Christoph Miethke GmbH & Co. KG names medical products consultants as the individuals to be addressed with all queries concerning the products:

Dipl.-Ing. Christoph Miethke  
Dipl.-Ing. Roland Schulz

Christoph Miethke GmbH & Co. KG  
Ulanenweg 2  
D-14469 Potsdam  
Tel.:+49(0) 7000 MIETHKE oder  
Tel.:+49 (0) 331 62083 0  
Fax:+49 (0) 331 62083 40  
e-mail: info@miethke.com

Please address any queries to:  
AESCULAP AG  
Am Aesculap-Platz  
D-78532 Tuttlingen  
Tel.:+49 (0) 7461 95 0  
Fax:+49 (0) 7461 95 26 00  
e-mail: information@aesculap.de

**General information**

Manufacturer	Christoph Miethke GmbH & Co. KG
Product name	<i>catheter</i>
Intended application	Treatment of hydrocephalus
Intended for one-time use only (disposable)	
Store in a clean and dry place	

**Variants**

The *catheters* are available as *ventricular catheters* and *peritoneal catheters*.

Inside Measurement: 1.2 mm  
Outside Measurement: 2.5 mm

**Ventricular catheter (adult and pediatric)**

Length 18 cm oder 25 cm

**Ventricular catheter with deflector (adult and pediatric)****Peritoneal catheter**

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Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Deutschland