


Recommendation of pressure levels

GAV[®] - GRAVITY ASSISTED VALVE



Age	Grade of activity	Height	Body shape	Recommended pressure level in cmH ₂ O
over 60 years old	mobile	less than 1,60 m	adipose	4/24 <i>paediGAV</i>
			slim	5/30
		over 1,60 m	adipose	5/30
			slim	5/35
	immobile	less than 1,60 m	adipose	4/24 <i>paediGAV</i>
			slim	4/24 <i>paediGAV</i>
		over 1,60 m	adipose	4/24 <i>paediGAV</i>
			slim	5/30
up to 60 years old	mobile	less than 1,60 m	adipose	4/24 <i>paediGAV</i>
			slim	5/30
		1,60 m - 1,80 m	adipose	5/30
			slim	5/35
		over 1,80 m	adipose	5/30
			slim	5/40
	immobile	less than 1,60 m	adipose	4/24 <i>paediGAV</i>
			slim	5/30
		over 1,60 m	adipose	4/24 <i>paediGAV</i>
			slim	5/30