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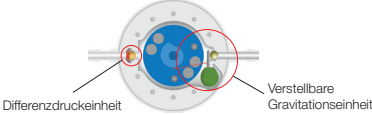
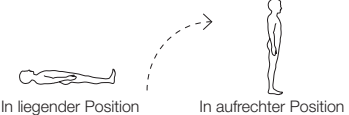
Patient	Selection of pressure levels		Total opening pressure	
				
Newborns and children up to 3 years 	5*	20	5*	25
Children from 3 years 		25		30
Adults between 1,60 m and 1,80 m height 		25		30
smaller than 1,60 m		20		25
larger than 1,80 m		30		35
Adults from 65 years 		20		25
smaller than 1,60 m		15		20
larger than 1,80 m		25		30

All indicated pressure levels are in cmH₂O. This is a non-binding recommendation for the attending physician. According to his diagnosis, the physician decides each case independently, without instructions and individually. The stated values consider the current scientific knowledge up to June 2018.
 * As special configurations in addition to the standard configuration the *M.blue* is also available with differential pressure units with pressure levels 0, 10, 15 cmH₂O. In case of *M.blue plus* the opening pressure of the differential pressure unit (*proGAV 2.0*) is adjustable from 0-20 cmH₂O.

M.blue[®] *M.blue plus*[®]

STANDARD PRESSURE LEVELS

The *M.blue* consists of a differential pressure unit and an adjustable gravitational unit.

The opening pressure of *M.blue* in the horizontal position is **only** determined by the differential pressure unit.

In the lying position the gravitational unit is not active.

The opening pressure in the vertical position is **the sum** of both: differential pressure unit and adjustable gravitational unit.

Our Strong Partner in Neurosurgery:

B | BRAUN
SHARING EXPERTISE

www.bbraun.com

The MIETHKE App demonstrates the functionality of *M.blue* and the effect of patient posture on valve opening pressure.

Now available in the App and Google Play Store.