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M.blue® M.blue plus®

STANDARD PRESSURE LEVELS*

Patient	Selection of pressure levels		Opening pressure in total	
				
	differential pressure unit	adjustable gravitational unit	horizontal position	upright position
Newborn 	5**	15 - 20	5	20 - 25
Small children from 6 months 	10**	15 - 20	10	25 - 30
Children from 3 years 	10	20 - 25	10	30 - 35
Adults 	5	25	5	30
< 1,60 m	5	20	5	25
> 1,80 m	5	30	5	35
Adults from 65 years 	5	20	5	25
< 1,60 m	5	15	5	20
> 1,80 m	5	25	5	30

The *M.blue* consists of a differential pressure unit and an adjustable gravitational unit.

The opening pressure of *M.blue* in the horizontal position is **only** determined by the differential pressure unit.

In the lying position the gravitational unit is not active.

The opening pressure in the vertical position is **the sum** of both: differential pressure unit and adjustable gravitational unit.

The Miethke App demonstrates the functionality of *M.blue* and the effect of patient posture on valve opening pressure.

Now available in the App and Google Play Store.

Our Strong Partner in Neurosurgery:

B | BRAUN
SHARING EXPERTISE

www.bbraun.com

All indicated pressure levels are in cmH₂O.

* This is a non-binding recommendation for the attending physician. According to his diagnosis, the physician decides each case independently, without instructions and individually. The stated values consider the current scientific knowledge up to 02/2021. ** In addition the *M.blue* is also available with differential pressure units with pressure levels 0 and 15 cmH₂O. In case of *M.blue plus* the opening pressure of the differential pressure unit (proGAV 2.0) is adjustable from 0-20 cmH₂O.

www.miethke.com