# SHUNTASSISTANT® 2.0

## SHUNTASSISTANT® 2.0 LP

### (SA® 2.0)

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*"We all have to carry our burden - some more so, others less.  
The most important thing is not to give up and to live your life."*

Sarah, nurse and hydrocephalus patient

## Foreword

Hydrocephalus is a disease that is frequently connected to the widest range of other existing problems. Some say that hydrocephalus is not a disease but the symptom of another underlying disorder.

Successful treatment of hydrocephalus only became possible in the 1950s. In a dramatic race for the life of his son Casey who had hydrocephalus, Philadelphia technician John D. Holder took only a few weeks to develop a silicone valve. The implantation of this valve in March 1956 marked a great step forward in the clinical treatment of this disease.

These days, the primary aim of hydrocephalus therapy is no longer to prevent death or severe impairment as the result of the condition, but to enable patients to leading a life that is as normal as possible.

Christoph Miethke GmbH & Co. KG has built on the insights of over 50 years of valve treatment, and with the development of gravitation technology pioneered the development of valves that consistently consider the physical framework conditions in the drainage of cerebrospinal fluid, thus ensuring an intracranial pressure in all body positions that matches as closely as possible that of a healthy person.

Their SA 2.0 is such a gravitational valve. The purpose of this handbook is to provide you and your family with a small insight into the treatment of hydrocephalus with the SA 2.0 implant.

## Contents

Basics	20
Hydrocephalus	20
SA 2.0 operating principle	22
Materials used in SA 2.0	27
Patient data card	27
Treatment complications	27
Conduct after the surgery	27
About the company	28

## Basics

The human brain is surrounded by a special fluid (scientific term: cerebro-spinal fluid - CSF). Inside the head are several chambers (scientific term: ventricles) where CSF is produced. These ventricles are interconnected with channels and constitute a complex drainage system.

CSF has the task of protecting the brain from mechanical damage, regulating intracranial pressure, maintaining moisture levels of the brain tissue and transporting metabolic products.

Fresh CSF is produced in the ventricles each day which is absorbed by the venous blood system. A healthy balance between the production and absorption of CSF is thus very important.

## Hydrocephalus

Hydrocephalus means that the balance between production and absorption of CSF is impaired. If more CSF is being produced than can be drained, the ventricles expand and a hydrocephalus develops ("hydro": Greek for water; "cephale": Greek for head). The ventricles exert pressure on the surrounding brain tissue and can cause neurological damage, some of which irreversible.

## Symptoms

Symptoms depend on the level of impairment and include headache, nausea, vomiting, coordination disorders, sleepiness up to the loss of consciousness. For children under the age of two, head circumference can increase hugely, because the bones in their skulls have not yet fused.

The causes of hydrocephalus are very varied and occur at any stage in life.

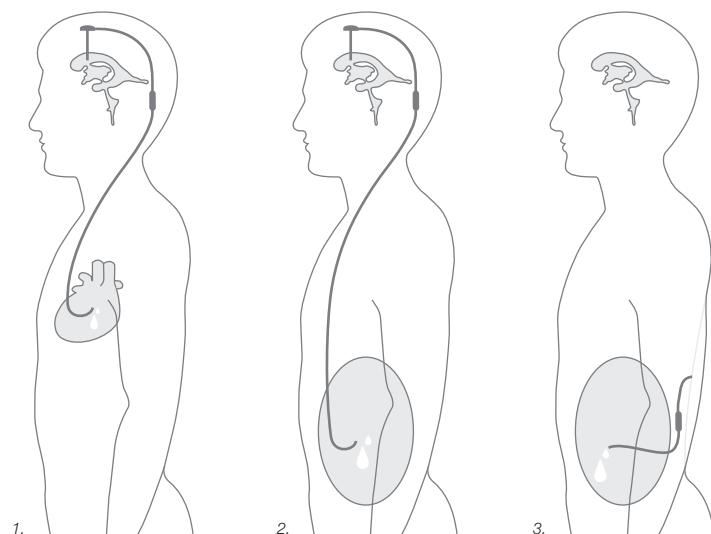
## Treatment methods

Even though there have been consistent efforts to find alternative treatment options to the implantation of a valve, e.g. by drug-based treatment or recently by minimally invasive surgical procedures, there is to this day not really an alternative in many cases to the implantation of a drainage system, known as a shunt.

In a hydrocephalus, intracranial pressure has to be reduced to within normal limits. To this end, in most cases a shunt system is implanted, creating a link between the ventricles and another bodily cavity - generally the abdomen - in order to drain off excess CSF from the ventricles.

There are three types of shunt implantation:

1. ventriculo-atrial (from the ventricle to the right atrium of the heart)
2. ventriculo-peritoneal (from the ventricle to the abdominal cavity)
3. lumbo-peritoneal (from the spinal canal to the abdominal cavity)


## SA 2.0 operating principle

A proper understanding of the operating principle of SHUNTASSISTANT 2.0 (SA 2.0) requires a brief look at the physical framework conditions that such drainage involves: creating a link between these two systems - ventricle and abdominal cavity - might allow the CSF to drain without control, particularly when the patient stands upright, thus transporting too much of the fluid from the ventricles to the abdominal cavity. This is called overdrainage and must be prevented by all means. However, it is entirely normal for a mobile human being to change position frequently: to stand up, walk, sit down or lie down. All this results in continuous physical changes in the drainage system.


In order to achieve a controlled drainage of the cerebrospinal fluid, a valve such as SA 2.0 is implanted together with the tube system. This combination of a tube system and a valve is known as a shunt system. Ideally, a valve such as SA 2.0 ensures that CSF is only drained off when the pressure inside the ventricles becomes too high. To that end, the valves have pre-set or settable opening pressures, i.e. a valve opens once a predefined intracranial pressure has been reached, thus preventing over- or underdrainage. However, the optimum opening pressure in a lying position is not the same as in an upright one. This is the consequence of the physical laws of gravity (gravitation).

### Physical background

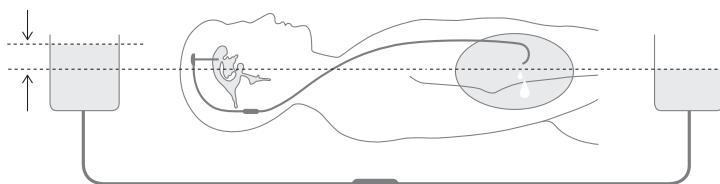Two interlinked fluid-filled vessels that are - as in a lying person - horizontal to each other aim to balance each other. A conventional differential pressure valve with a low opening pressure requires a second component with a higher opening pressure in the upright position, as otherwise too much cerebrospinal fluid can drain in an upright body position. This could lead to excessive drainage and therefore to serious complications.


If you now simulate a patient standing up by holding the vessel simulating the ventricles much higher than that representing the abdominal cavity, the fluid tries to follow gravity and drain downwards. The height of a patient has a significant impact on the length of the water column being created in the catheter in the upright position, and also on the suction effect. The valve previously set to a low pressure now opens at a low opening pressure in the upright position, too, thus ensuring that excessive cerebrospinal fluid drains down while the patient is upright.

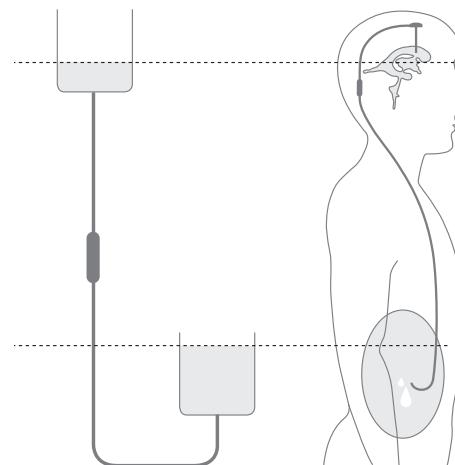A single opening pressure would be a constant compromise between the lying and the upright body position and would constitute a serious restriction to the quality of life.

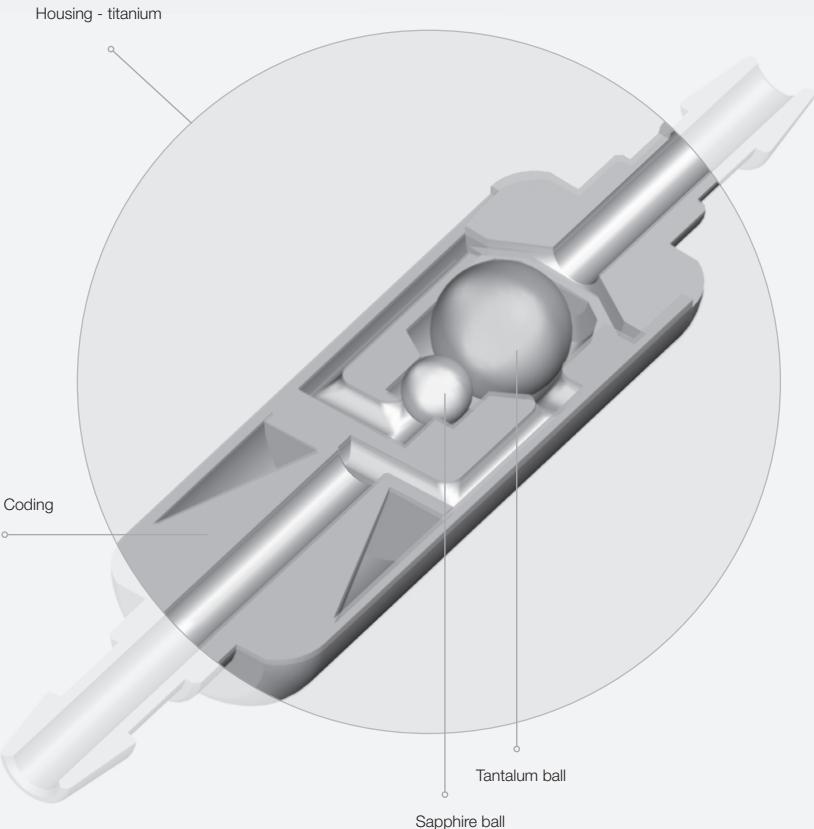This means that different opening pressures are required for the various body positions. As a result, an SA 2.0 is offered as an additional valve in order to provide a higher opening pressure in an upright body position.

## SA 2.0 mechanism of action

SA 2.0 is a gravitational valve which makes use of gravity in order to achieve an intracranial pressure independent of the body position of the patient that is guided by pressure values of healthy subjects.

The SA 2.0 contains a gravitational unit, which can be combined with conventional differential pressure valves. The SA 2.0 has a pre-set opening pressure, selected by the neurosurgeon prior to implantation.

### Differential pressure valve

A conventional differential pressure valve operates the same in all body positions; for that reason, it has generally got a low opening pressure set for the horizontal position to ensure that in case of increasing intracranial pressure the valve permits the timely drainage of cerebrospinal fluid particularly in the horizontal position.

### Gravitational unit

The gravitational unit makes use of the principle of gravity and operates with the aid of a heavy tantalum ball in the inside of SA 2.0. In the upright position, this ball presses down on a cone, thus closing the valve until a specific opening pressure has been reached, which then dislocates the ball from its position, thus permitting the drainage of cerebrospinal fluid. If the intracranial pressure drops back below the pre-set opening pressure, the ball once again closes the valve.

In the upright position, the gravitational unit operates in addition to the differential pressure unit. In that way, SA 2.0 achieves in upright position the desired opening pressure.

**Warning notice:** The shunt system may be fitted with a pumpable reservoir. As too frequent pumping can result in an excessive drainage (overdrainage) and thus unfavourable pressure conditions, this process should only be performed by a physician.

## Materials used in SA 2.0

SA 2.0 consists of materials that have been proven and standardises for use in implants; the main component is titanium. The stable housing ensures the reduction of outside influences (such as external pressure) on the valve function. Thus a high functional reliability and a long service life can be reached.

Catheters are made from silicone and are not made with natural rubber latex.

## Patient data card

Each SA 2.0 is accompanied by a patient data card. This is completed by the treating physician and thus contains all of the important information for follow-up examinations.

## Treatment complications

The treatment of hydrocephalus with a shunt system is not always free from complications. As with every surgical intervention, there is a risk of infection.

Unfortunately, problems can develop occasionally that could be directly or indirectly linked to the implanted valve. Such complications include blockages of the shunt system or the undesired excessive or insufficient drainage of cerebrospinal fluid (over- or underdrainage).

## Conduct after the surgery

Patients supplied with a shunt system are normally not restricted in their daily lives. They can travel by plane with the SA 2.0, do sports or go to work.

Only for the period immediately after the surgery, you should discuss with your physician to what extent you should avoid physical exertion and for how long that would be advisable in your specific case.

For MRI follow-up examinations: SA 2.0 is MR Conditional up to 3 Tesla.

## About the company

Christoph Miethke GmbH & Co. KG, based in Potsdam, has from 1992 concentrated on the development, production and distribution of innovative neurosurgical implants for the treatment of hydrocephalus.

Our daily aim is through the innovative development and production of high-quality reliable products to improve the life of hydrocephalus patients. One of our foremost tasks hereby in cooperation with hospitals, physicians and patients is primarily this: to understand. Again and again, we change perspectives and meet with our colleagues, partners and patients on an equal footing.

For detailed information about our company, please visit our website:  
[www.miethke.com](http://www.miethke.com)

More about the operating principle of our valves is available here:  
[www.miethke.com/products](http://www.miethke.com/products)

Or use our app:  
Apple: <https://itunes.apple.com/de/app/miethke/id450290015?mt=8>  
Android: <https://play.google.com/store/apps/details?id=com.miethke.graviton>

You are also welcome to contact our Medical Devices Consultants:  
+49 331 620 83-0 // [info@miethke.com](mailto:info@miethke.com) // [www.miethke.com](http://www.miethke.com)

Roland Schulz

Michaela Funk-Neubarth

Josefine Kehl

Thammo Weise

August von Hardenberg

Andreas Bunge

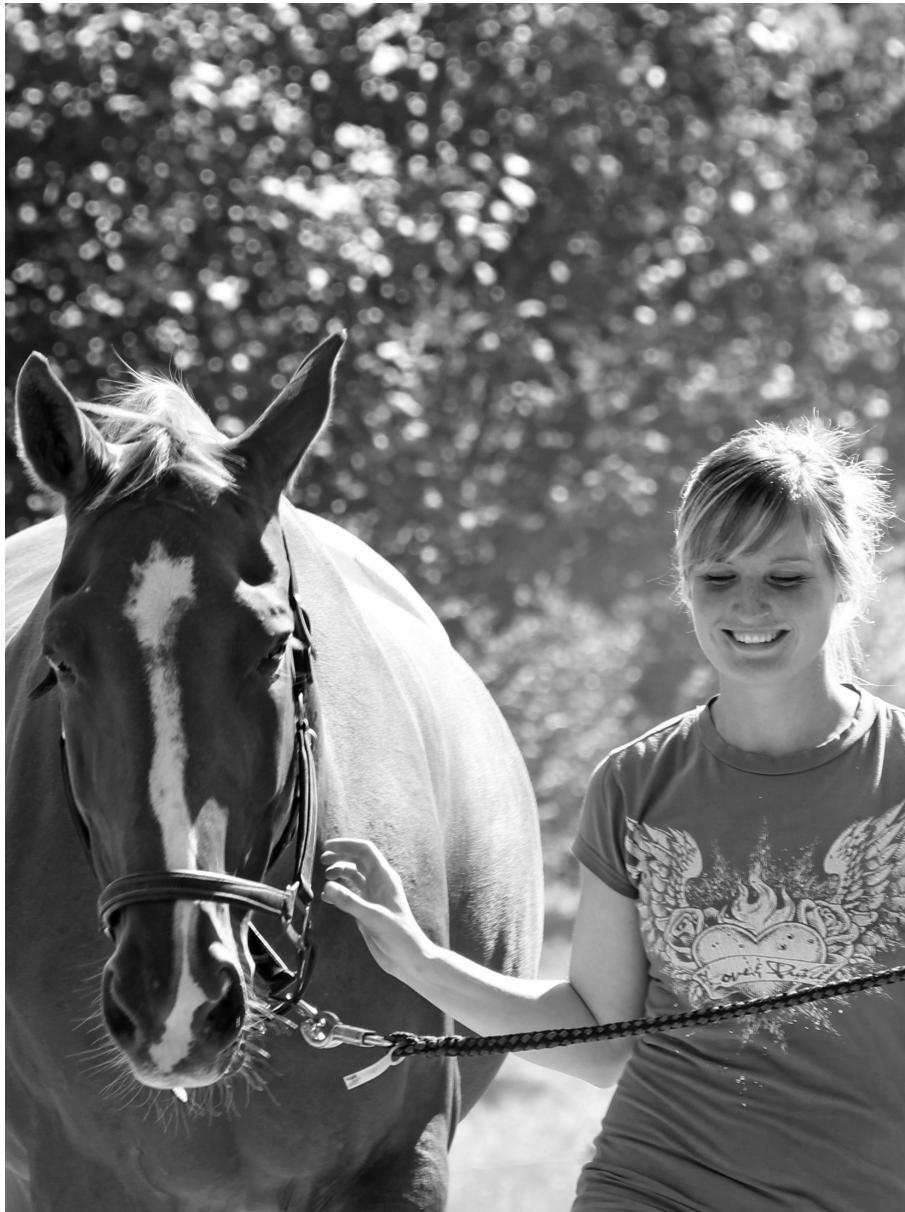
Jan Mügel

Thoralf Knitter

For up-to-date information, visit our social media channels:

[facebook.com/MiethkeDeutschland](https://facebook.com/MiethkeDeutschland) [youtube.com/Miethke\\_International](https://youtube.com/Miethke_International)

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Christoph Miethke GmbH & Co. KG | Ulanenweg 2 | 14469 Potsdam | Germany
Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany