# *SENSOR RESERVOIR® / SENSOR PRECHAMBER®*

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**CONTENTS**

INDICATION	8
INTENDED PURPOSE	8
TECHNICAL DESCRIPTION	8
MODE OF OPERATION	8
FUNCTIONAL TESTS	9
IMPLANTATION	10
PRECAUTIONS AND CONTRAINDICATIONS	10
FUNCTIONAL SAFETY	10
COMPATIBILITY WITH DIAGNOSTIC AND THERAPEUTIC PROCEDURES	10
SIDE EFFECTS	11
STERILISATION	11
SHELF LIFE   STORAGE	11
PATIENT PASSPORT AND SD CARD	11
REQUIREMENTS OF THE AIMDD DIRECTIVE (90/385/EEC)   CE MARK	11
MEDICAL DEVICES CONSULTANT	11

## INDICATION

The *SENSOR RESERVOIR / SENSOR PRECHAMBER* serves as a reservoir to remove cerebrospinal fluid (CSF) and administer medication in the treatment of hydrocephalus. In addition, the reservoirs can be used for the pressure-based functional control of the shunt system.

## INTENDED PURPOSE

The *SENSOR RESERVOIR / SENSOR PRECHAMBER* is used for the pressure-based functional control of shunt system.

## TECHNICAL DESCRIPTION

One option in the treatment of hydrocephalus is the implantation of a shunt system, which facilitates the drainage of the cerebrospinal fluid from the ventricles into a suitable part of the body (typically the abdominal area). Such a shunt system consists of multiple catheters and a valve. In addition, a *SENSOR RESERVOIR / SENSOR PRECHAMBER* can be integrated into the shunt system, thus fulfilling the function of a conventional reservoir, i.e. it offers the opportunity to measure intraventricular pressure, to inject drugs and to control the functioning of the valve.

The robust titanium housing of the integrated measuring cell prevents any accidental puncturing of the base. Taps should be performed perpendicular to the reservoir surface with a cannula of max.  $\varnothing$  0.9 mm. 30 taps are possible without any restrictions. The volume per pump procedure is approx. 0.062 ml. The use of the *SENSOR RESERVOIR / SENSOR PRECHAMBER* does not increase the opening pressure of the shunt system.

In addition, the *SENSOR RESERVOIR / SENSOR PRECHAMBER* (Fig. 1) offers the option of performing a non-invasive functional control of the shunt system. To this end, knowledge of the pressure inside the reservoir and of pressure changes is of particular interest. Integrating the measuring cell into the reservoir makes it possible to measure and record these pressures; the relative pressure readings thus gained can then be used to draw conclusions as to the proper functioning of the shunt system.

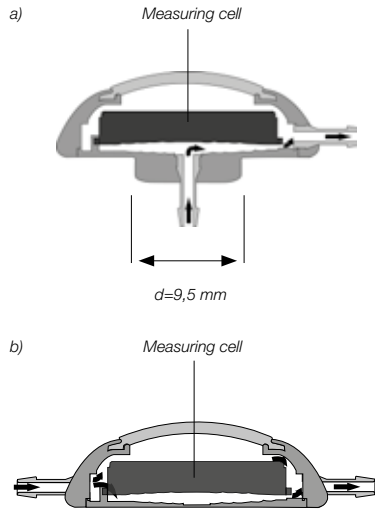
Fig. 1: Functional principle with integral measuring cell  
a) *SENSOR RESERVOIR*  
b) *SENSOR PRECHAMBER*

## MODE OF OPERATION

With the reader unit set for the *SENSOR RESERVOIR / SENSOR PRECHAMBER* (henceforth referred to as "reader unit"), measured data from the measuring cell in the *SENSOR RESERVOIR / SENSOR PRECHAMBER* are read and displayed. Measured data are automatically stored on the SD card for later evaluation. The relative behaviour of the CSF pressure in the shunt system can provide information about its function. This permits the non-invasive detection, localisation and evaluation of an occlusion within the shunt system as well as a mechanical loss of function.

**Caution: Looking at these relative pressure readings in isolation cannot be taken as reliable data for absolute brain pressure. Even though the pressure metering in the reader unit is monitored regularly, thus complying with all criteria of calibrated assessment, the same is not possible for the implanted cell.**

If the shunt system is to be revised on the basis of data gained from the implant with integrated measuring cell, the diagnosis must be

confirmed through combination with other non-invasive, but also interventional and radiological diagnostic methods (CT, MRI, tap or another method).

Fig. 2: Measuring pressure with the *SENSOR RESERVOIR / SENSOR PRECHAMBER* reader unit

## FUNCTIONAL TESTS

### Functional test prior to implantation

**Caution: The functionality of the implant must be tested prior to implantation.**

The reader unit and the SD card associated with the implant are used to test whether the measuring cell can be addressed correctly (see instructions for use for the *SENSOR RESERVOIR* reader unit set). If the sensor ID (individual sensor cell number) is recognised by the reader unit, measured data can be read correctly. The *SENSOR RESERVOIR / SENSOR PRECHAMBER* should be checked for patency prior to implantation. Patency can be checked by aspiration of a sterile fluid from the distal end.

### Post-implantation functional test

After implantation, the functionality of the implant should be checked once again with the aid of the reader unit and the associated SD card (see instructions for use for the *Reader Unit Set for SENSOR RESERVOIR*). If communication between the measuring cell and the reader unit is established, a simple plausibility test can be performed with the patient in a stable position: slight pressure is exerted on the membrane of the *SENSOR RESERVOIR / SENSOR PRECHAMBER* and the increase in pressure displayed on the reader unit.

The measuring cell is calibrated. Its correct functioning is guaranteed under the following conditions:

Temperature range	20-39°C
Pressure measuring range (relative)	-66,66 hPa -133,32 hPa (-50 - 100 mmHg)
Pressure measuring range (absolute)	800-1100 hPa

### Warning notices:

Conclusions about absolute pressure: In patients with acute or chronic brain disorders or injuries (e.g. craniocerebral trauma, cerebral haemorrhage, hydrocephalus, brain tumours, etc.), please note that the measuring cell can only be used to obtain differential pressure measurements (relative, time-dependent changes in CSF values) when taking cerebral pressure readings. The readings taken by the measuring cell do not permit conclusions as to the absolute pressure within the cranial cavity. To determine such absolute pressure values, reference pressure has to be assessed transcutaneously.

Excessive pumping: Frequent pumping of the *SENSOR RESERVOIR / SENSOR PRECHAMBER* can result in excessive drainage, thus leading to non-physiological pressure ratios. The patient should be informed about this risk.

Raised temperature: If the patient has a raised temperature, functional impairment of the reader unit may occur (see instructions for use for the *Reader Unit Set for SENSOR RESERVOIR*). In readout mode, a rise in temperature in the *SENSOR RESERVOIR / SENSOR PRECHAMBER* may result. An integrated temperature safety device stops measurements at 39 °C.

High pressure and shock load: If high pressure or physical shocks are likely from patient activities (diving, boxing, football, etc.), a *SENSOR RESERVOIR / SENSOR PRECHAMBER* should not be used in shunt systems.

## IMPLANTATION

The *SENSOR RESERVOIR / SENSOR PRECHAMBER* must always be placed outside of the skullcap. A burr hole with a diameter of 10 mm is recommended for the implantation of a *SENSOR RESERVOIR*. The *SENSOR RESERVOIR / SENSOR PRECHAMBER* is designed for use with catheters with an internal diameter of approx. 1.2 mm and an external one of approx. 2.5 mm. The ventricular catheter is implanted with the help of a mandrel. The implantation of the *SENSOR PRECHAMBER* also requires a burr hole deflector to orient the ventricular catheter at a 90° angle. After connection to the *SENSOR RESERVOIR / SENSOR PRECHAMBER*, the catheter must be secured using a ligature. The *SENSOR RESERVOIR / SENSOR PRECHAMBER* is connected to the complete shunt system. When implanting a *SENSOR RESERVOIR / SENSOR PRECHAMBER* without a distal catheter, care should be taken to ensure that the chosen drainage catheter is also secured using a ligature. We recommend only to use Miethke products in combination with the *SENSOR RESERVOIR / SENSOR PRECHAMBER*. The position of the ventricular catheter should be checked after the procedure by CRT or MRI. It is recommended to check the entire shunt system for patency.

Products that have previously been implanted must not subsequently be reimplanted into the same or another patient.

## PRECAUTIONS AND CONTRAINDICATIONS

Patients must be carefully monitored after implantation. Reddening of skin or tightness in the area of the drained tissue may be indications of infections at the shunt system. Symptoms such as headache, dizziness, confusion or vomiting often occur in conjunction with shunt dysfunction. These symptoms and a leakage within the shunt system require the immediate replacement of the affected shunt component or the entire shunt system.

The implantation of medical devices is contraindicated if the patient has an infection or suspected infection (e.g. meningitis, ventriculitis, peritonitis, bacteraemia, septicaemia) in the region affected by the implantation.

## FUNCTIONAL SAFETY

These medical devices are constructed in such a way as to ensure their precise and reliable operation over long periods of time. However, no guarantee can be given that these medical devices may not require replacement for medical or technical reasons. The *SENSOR RESERVOIR / SENSOR PRECHAMBER* as well as the entire shunt system are safely able to resist positive and negative pressures up to 100 mmHg during and after the procedure. If the measuring cell fails, the *SENSOR RESERVOIR / SENSOR PRECHAMBER* continues to function as a conventional burr hole reservoir or respectively prechamber, without any limitation. The integrated measuring cell poses no additional risk. These medical devices have to be stored in a clean and dry environment at all times.

## COMPATIBILITY WITH DIAGNOSTIC AND THERAPEUTIC PROCEDURES

The *SENSOR RESERVOIR / SENSOR PRECHAMBER* and the distal catheter (if present) are made from the non-magnetic substances PEEK, titanium and silicone. MRI or CT scans can be performed without impairment up to a field strength of 3 tesla. Artefacts can occur in MRI scans. The *SENSOR RESERVOIR / SENSOR PRECHAMBER* are MRI Conditional. The catheters supplied with the product are MRI Safe.

### Warning notices:

**The use of energy-emitting devices such as defibrillators and HF devices can cause the measuring cell to fail!**

**In cases where an electrical current from an external source comes into contact with the body, the measuring cell can sustain damage.**

**The use of radiation therapy and radionuclide patient imaging procedures can cause the measuring cell to fail.**

**During therapeutic ultrasound procedures, there is a risk of the concentration of the ultrasound field being unintentionally increased and thus injuring the patient.**

## SIDE EFFECTS

In the treatment of hydrocephalus with shunts, the following complications may arise (as described in the literature): Infections, blockages caused by protein and/or blood in the cerebrospinal fluid, over/under drainage or in very rare cases noise development. Violent shocks for the outside (accident, fall) may put the integrity of the shunt system at risk.

## STERILISATION

The products are sterilised under strictly controlled conditions. The double wrapping in sterile bags ensures sterility for a five-year period. The expiry date is printed on the wrapping of each individual product. If the packaging is damaged, the product must not be used in any circumstances. No guarantee can be given for the functional safety and reliability of resterilised products.

## SHELF LIFE | STORAGE

The manufacturer guarantees the functionality (lifetime) of the measuring cell integrated into the *SENSOR RESERVOIR / SENSOR PRECHAMBER* for three years. The expiry date is printed on the package. The function of the *SENSOR RESERVOIR / SENSOR PRECHAMBER* as a pure burrhole reservoir or prechamber is not influenced by the measuring cell and continues safely for more than three years.

Temperature range for storage	0°C-50°C
Pressure range for storage	800hPa-1100hPa

## PATIENT CARD AND SD CARD

The treating physician is encouraged to fill in the complete patient card. In addition to the patient card, the patient is given an SD card on which all individual data about the implant are stored. If the SD card is lost, it can be reordered by specifying the serial number of the SD card.

## REQUIREMENTS OF THE AIMDD DIRECTIVE (90/385/EEC) | CE MARK

The Medical Device Directive requires the comprehensive documentation of the whereabouts of medical devices used in humans. The individual identification number of the implanted valve should therefore be recorded in the patient's medical records and patient card to ensure complete traceability. Approval for marking with the CE mark for active implantable medical devices (in accordance with Directive 90/385/EEC) was granted for the first time in 2011.

## MEDICAL DEVICES CONSULTANT

In compliance with the AIMDD directive (90/385/EEC), Christoph Miethke GmbH & Co. KG has nominated medical product consultants as contacts for all product-related questions:

Dipl.-Ing. Christoph Miethke  
Dipl.-Ing. Roland Schulz  
Michaela Funk-Neubarth

Contact details can be found on the reverse of these instructions for use.

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Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany