proGAV[®] 2.0

IN TOUCH WITH YOU

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
 facebook.com/MiethkeDeutschland  youtube.com/Miethke_International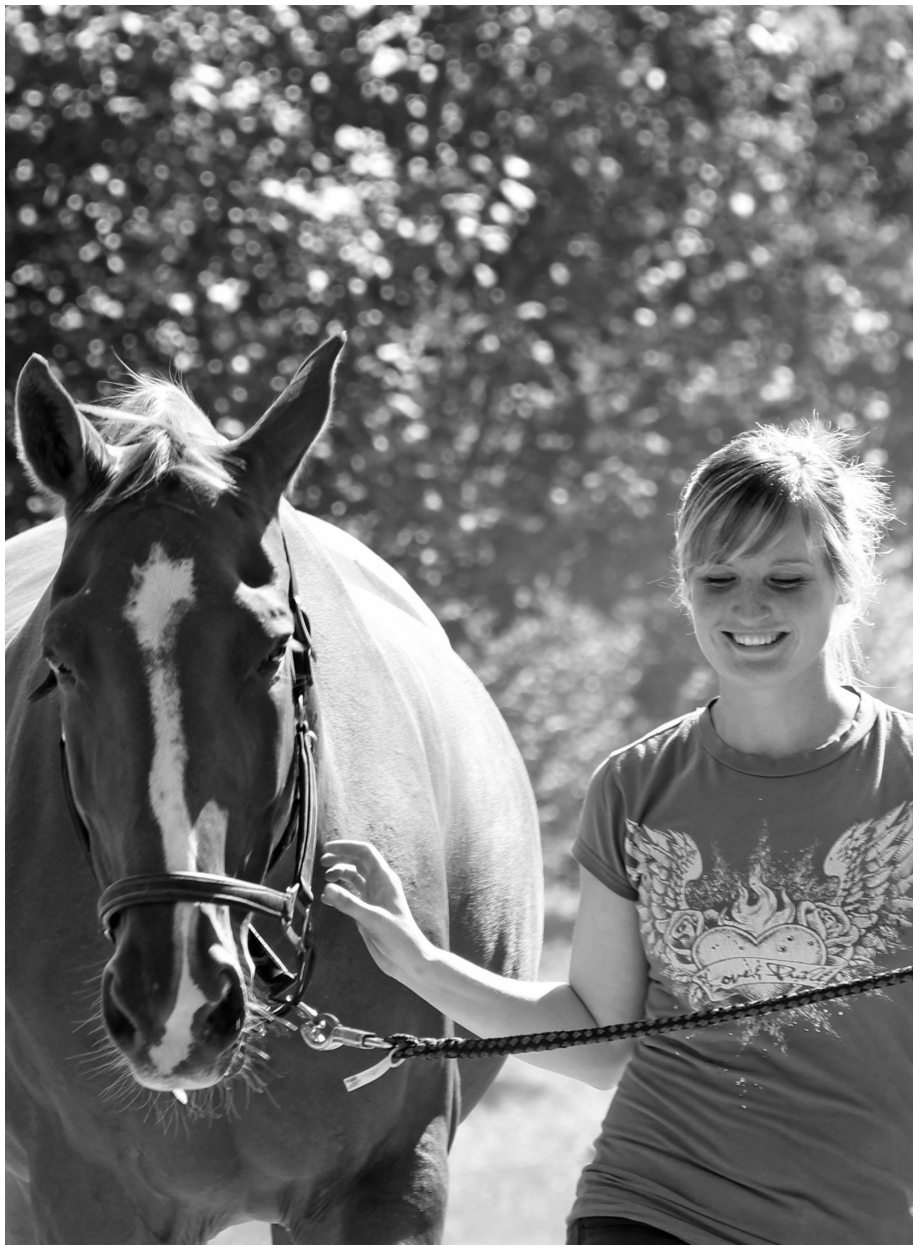

*"We all have to carry our burden - some more so, others less.
The most important thing is not to give up and to live your life."*

Sarah, nurse and hydrocephalus patient

Foreword

Hydrocephalus is a disease that is frequently connected to the widest range of other existing problems. Some say that hydrocephalus is not a disease but the symptom of another underlying disorder.

Successful treatment of hydrocephalus only became possible in the 1950s. In a dramatic race for the life of his son Casey who had hydrocephalus, Philadelphia technician John D. Holder took only a few weeks to develop a silicone valve. The implantation of this valve in March 1956 marked a great step forward in the clinical treatment of this disease.

These days, the primary aim of hydrocephalus therapy is no longer to prevent death or severe impairment as the result of the condition, but to enable patients to leading a life that is as normal as possible.

Christoph Miethke GmbH & Co. KG has built on the insights of over 50 years of valve treatment, and with the development of gravitation technology pioneered the development of valves that consistently consider the physical framework conditions in the drainage of cerebrospinal fluid, thus ensuring an intracranial pressure in all body positions that matches as closely as possible that of a healthy person.

Their *proGAV 2.0* is such a gravitational valve. The purpose of this handbook is to provide you and your family with a small insight into the treatment of hydrocephalus with the *proGAV 2.0* implant.

Contents

Basics	20
Hydrocephalus	20
<i>proGAV 2.0</i> operating principle	22
Materials used in <i>proGAV 2.0</i>	27
Patient data card	27
Treatment complications	27
Conduct after the operation	27
About the company	28

Basics

The human brain is surrounded by a special fluid (scientific term: cerebrospinal fluid - CSF). Inside the head are several chambers (scientific term: ventricles) where CSF is produced. These ventricles are interconnected with channels and constitute a complex drainage system.

CSF has the task of protecting the brain from mechanical damage, regulating intracranial pressure, maintaining moisture levels of the brain tissue and transporting metabolic products.

Fresh CSF is produced in the ventricles each day which is absorbed by the venous blood system. A healthy balance between the production and absorption of CSF is thus very important.

Hydrocephalus

Hydrocephalus means that the balance between production and absorption of CSF is impaired. If more CSF is being produced than can be drained, the ventricles expand and a hydrocephalus develops ("hydro": Greek for water; "cephale": Greek for head). The ventricles exert pressure on the surrounding brain tissue and can cause neurological damage, some of which irreversible.

Symptoms

Symptoms depend on the level of impairment and include headache, nausea, vomiting, coordination disorders, sleepiness up to the loss of consciousness. For children under the age of two, head circumference can increase hugely, because the bones in their skulls have not yet fused.

The causes of hydrocephalus are very varied and occur at any stage in life.


Treatment methods

Even though there have been consistent efforts to find alternative treatment options to the implantation of a valve, e.g. by drug-based treatment or recently by minimally invasive surgical procedures, there is to this day not really an alternative in many cases to the implantation of a drainage system, known as a shunt.

In a hydrocephalus, intracranial pressure has to be reduced to within normal limits. To this end, in most cases a shunt system is implanted, creating a link between the ventricles and another bodily cavity - generally the abdomen - in order to drain of excess CSF from the ventricles.

There are three types of shunt implantation:

1. ventriculo-atrial (from the ventricle to the right atrium of the heart)
2. ventriculo-peritoneal (from the ventricle to the abdominal cavity)
3. lumbo-peritoneal (from the spinal canal to the abdominal cavity)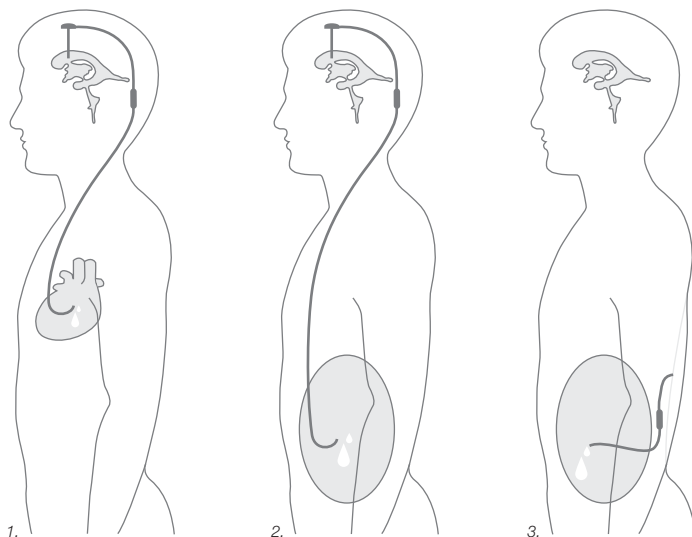


proGAV 2.0 operating principle

A proper understanding of the operating principle of *proGAV 2.0* requires a brief look at the physical framework conditions that such drainage involves: creating a link between these two systems - ventricle and abdominal cavity - might allow the CCSF to drain without control, particularly when the patient stands upright, thus transporting too much of the fluid from the ventricles to the abdominal cavity. This is called overdrainage and must be prevented by all means. However, is it entirely normal for a mobile human being to change position frequently: to stand up, walk, sit down or lie down. All this results in continuous physical changes in the drainage system.


In order to achieve a controlled drainage of the cerebrospinal fluid, a valve such as *proGAV 2.0* is implanted together with the tube system. This combination of a tube system and a valve is known as a shunt system. Ideally, a valve such as *proGAV 2.0* ensures that CSF is only drained off when the pressure inside the ventricles becomes too high. To that end, the valves have pre-set or settable opening pressures, i.e. a valve opens once a predefined intracranial pressure has been reached, thus preventing over- or underdrainage. However, the optimum opening pressure in a prone position is not the same as in an upright one. This is the consequence of the physical laws of gravity (gravitation).

Physical background

Two interlinked fluid-filled vessels that are - as in a prone person - horizontal to each other aim to balance each other. If the valve has the same opening pressure in the upright position, too much fluid will drain. This equates to a low opening pressure.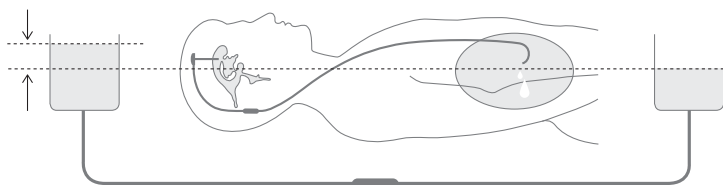


If you now simulate a patient standing up by holding the vessel simulating the ventricles much higher than that representing the abdominal cavity, the fluid tries to follow gravity and drain downwards. The height of a patient has a significant impact on the length of the water column being created in the catheter in the upright position, and also on the suction effect. The valve previously set to a low pressure now opens at a low opening pressure in the upright position, too, thus ensuring that excessive cerebrospinal fluid drains down while the patient is upright.

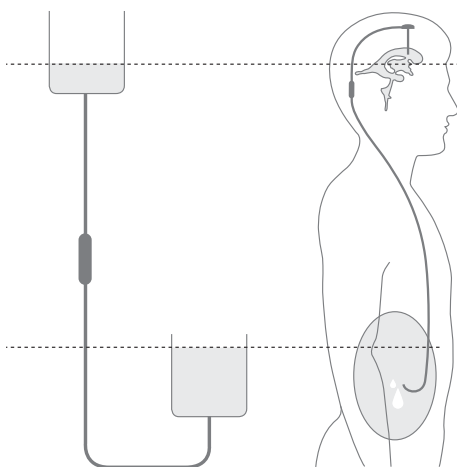A single opening pressure would be a constant compromise between the prone and the upright body position and would constitute a serious restriction to the quality of life.

This means that different opening pressures are required for the various body positions.

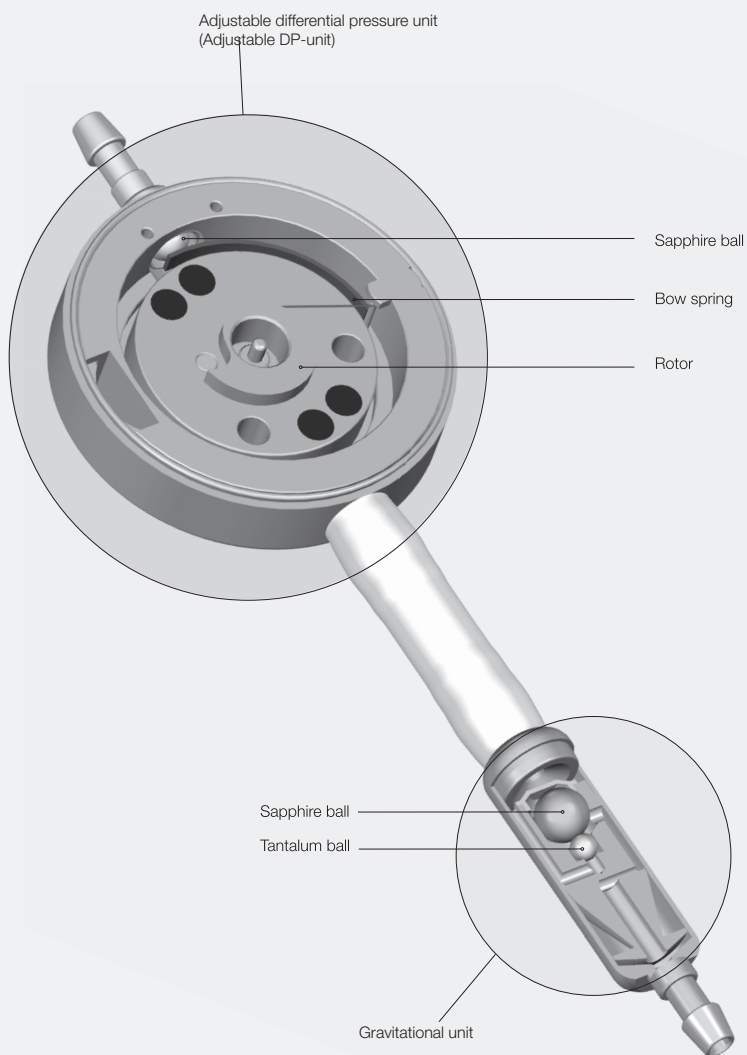proGAV 2.0 mechanism of action

The *proGAV 2.0* is a position-dependent valve. It consists of an adjustable differential pressure unit and a gravitational unit. In this way it can be ensured that optimal CSF drainage is maintained for every individual patient, independent of the body position of the patient.

Adjustable differential pressure unit

The adjustable differential pressure unit operates the same in all body positions; for that reason, it has generally got a low opening pressure set for the horizontal position to ensure that in case of increasing intracranial pressure the valve permits the timely drainage of cerebrospinal fluid particularly in the horizontal position. This opening pressure can be changed post-operative through the skin, without necessitating any further surgical intervention.

Gravitational unit

The gravitational unit makes use of the principle of gravity. As soon as the patient moves to an upright position, a tantalum ball is activated, which, by its weight, sets the opening pressure of the valve to a higher value. The closer the upperbody of the patient is to the vertical position, the higher the total opening pressure of the *proGAV 2.0*.

Adjustment mechanism

The *proGAV 2.0* is an adjustable valve. Unintended readjustments by external magnet fields are protected by the brake of the rotor. The adjustable differential pressure unit of *proGAV 2.0* is equipped with a feedback mechanism. Due to the unique construction of the valve housing, pressuring on the valve produces an acoustic signal (a clicking sound) which is audible or noticeable as a tactile resistance, so the brake of the rotor is now released. The valve indicates acoustically and haptically that the brake of the rotor is released and the pressure is sufficient to decoupling. The clicking sound is well recognizable before implantation. However after implantation, once the valve is filled up, depending on place and texture of the surrounding area of the implant, the acoustic signal could be considerably muted. The clicking sound should generally be audible by the patient itself or via a stethoscope.

Warning note: Applying pressure on the feedback-mechanism is restricted to the physician.

Warning note: The shunt system may be fitted with a pumpable reservoir. As too frequent pumping can result in an excessive drainage (overdrainage) and thus unfavourable pressure conditions, this process should only be performed by a physician.

Materials used in *proGAV 2.0*

proGAV 2.0 consists of materials that have been proven and standardises for use in implants; the main component is titanium. The stable housing ensures the reduction of outside influences (such as external pressure) on the valve function. Thus a high functional reliability and a long service life can be reached.

Catheters are made from silicone and are not made with natural rubber latex.

Patient data card

Each *proGAV 2.0* is accompanied by a patient data card. This is completed by the treating physician and thus contains all of the important information for follow-up examinations.

Treatment complications

The treatment of hydrocephalus with a shunt system is not always free from complications. As with every surgical intervention, there is a risk of infection.

Unfortunately, problems can develop occasionally that could be directly or indirectly linked to the implanted valve. Such complications include blockages of the shunt system or the undesired excessive or insufficient drainage of cerebrospinal fluid (over- or underdrainage).

Conduct after the operation

Patients supplied with a shunt system are normally not restricted in their daily lives. They can travel by plane with the *proGAV 2.0*, do sports or go to work.

Only for the period immediately after the surgery, you should discuss with your doctor to what extent you should avoid physical exertion and for how long that would be advisable in your specific case.

For MRI follow-up examinations: *proGAV 2.0* is MR Conditional up to 3 Tesla.

About the company

Christoph Miethke GmbH & Co. KG, based in Potsdam, has from 1992 concentrated on the development, production and distribution of innovative neurosurgical implants for the treatment of hydrocephalus.

Our daily aim is through the innovative development and production of high-quality reliable products to improve the life of hydrocephalus patients. One of our foremost task hereby in cooperation with hospitals, physicians and patients is primarily this: to understand. Again and again, we change perspectives and meet with our colleagues, partners and patients on an equal footing.

For detailed information about our company, please visit our website:
www.miethke.com

More about the operating principle of our valves is available here:
<https://www.miethke.com/en/products/>

Or use our app:

Apple: <https://itunes.apple.com/de/app/miethke/id450290015?mt=8>

Android:

<https://play.google.com/store/apps/details?id=com.miethke.graviton>

You are also welcome to contact our Medical Devices Consultants:
+49 331 620 83-0 // info@miethke.com // www.miethke.com

Roland Schulz

Michaela Funk-Neubarth

Josefine Kehl

Thammo Weise

August von Hardenberg

Andreas Bunge

Jan Mügel

Thoralf Knitter

For up-to-date information, visit our social media channels:

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Christoph Miethke GmbH & Co. KG | Ulanenweg 2 | 14469 Potsdam | Germany
Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany